Feelo.

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Producent: Shenzhen Jermei Medical Device Technology Co., Ltd. Room 201, 202,
4th Bldg XianYuXing Company, 4th Bldg XianYuXing industrial Zone, Yuhe Road,
Gonghe Community, Shajing Street, Baoan District, Shenzhen, P.R. China

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Set Elements

- 1 nebulizer with medication cup
- 1 adult mask
- 1 child mask
- 1 mouthpiece
- 1 USB-C charging cable



Description of the Ultrasound Nebulizer

The Feelo ultrasound nebulizer is a medical device. It is a reliable solution that will help you take care of the health of your entire family. It is used for administering inhaled medications directly into the respiratory tract. The device uses piezoelectric mesh spray technology, which transforms liquid medication into a fine mist. Thanks to microscopic holes in the mesh, the liquid is sprayed into very fine particles (medication particles sized 2-5 μm), which are directly transported to the upper and lower respiratory tracts. It is intended for both adults and children. Children should use the product under adult supervision. For children under 3 years of age, the use of the nebulizer should be consulted with a doctor.

Features and Functions

- The nebulizer features high efficiency, minimizing the amount of residual medication to below 0.1 ml, ensuring the full utilization of every drop.
- Ultra-quiet operation (<30 dB) allows for inhalation at any time without disturbing household peace.
- The built-in battery provides up to 120 minutes of operation on a single charge.
- The nebulizer offers two spray speeds, allowing the intensity of therapy to be adjusted to the user's individual needs.
- The nebulizer is lightweight and compact, facilitating the administration of inhaled medications, even while traveling. Its small size makes it ideal for conducting inhalations for young children (under adult supervision).
- The nebulizer can be used with most inhaled medications available at the pharmacy.
- Ultrasound nebulizers are not suitable for nebulizing suspensions or oil-based preparations, as they can clog the device and do not form the proper mist.
- Ultrasound nebulizers are not intended for nebulizing powdered medications. Powders must first be dissolved in an appropriate solution.
- Always consult a doctor before using a new medication in an ultrasound nebulizer. The doctor can confirm whether the medication is suitable for nebulization and at what dosage it should be used.
- Always follow the manufacturer's instructions for the use of medications in the nebulizer.

Components of the Nebulizer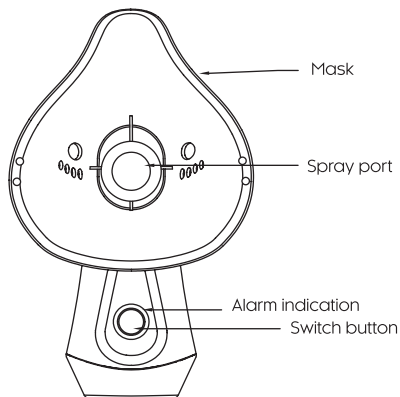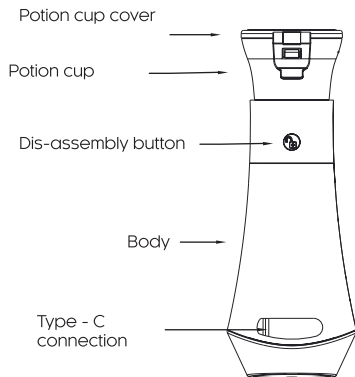

Charging

Connect the nebulizer to a power source using the USB-C cable to a USB charger to start charging. Before first use, the device should be fully charged. Full charging takes approximately 3-4 hours. To extend battery life, charge the device when it indicates the need for charging. If the device will not be used for an extended period, charge it fully before storage.

Important: Use the appropriate power adapter. Please ensure that the power adapter used has output parameters that match those specified on the label of your nebulizer. Using an adapter with incompatible parameters may lead to improper device operation and is absolutely unacceptable. Failure to comply with this rule may result in device damage and loss of warranty. Take care of your device by using only recommended accessories and power adapters. We recommend using a power adapter with specifications of 5V - 1A.

Alarm Indication

- Flashing yellow light: low battery
- Flashing green light: charging
- Green light: charge complete

Using the Nebulizer

Before Each Use

- Ensure that the nebulizer components are hygienically clean and dry. More information on cleaning the nebulizer can be found in the chapter: Cleaning and Storage.
- Make sure the nebulizer is charged.

Filling the Medication Cup


- Attach the medication cup to the device body.
- Open the medication cup lid and pour the medication until it fills $\frac{2}{3}$ of the volume. The maximum amount of medication is 15 ml.
- Close the medication cup lid.

Medication dosage: Use the amount of medication as recommended by your doctor or as indicated in the medication leaflet. Following the recommendations is crucial for the effectiveness and safety of the therapy.

Important: Do not use distilled water, essential oils, or sticky liquids in the device. Never turn on the nebulizer if the medication cup is empty.

Attaching the Mask/Mouthpiece

Ensure that the medication cup is connected to the nebulizer and filled with medication, then attach the silicone mask/mouthpiece to the device as shown in the figure below.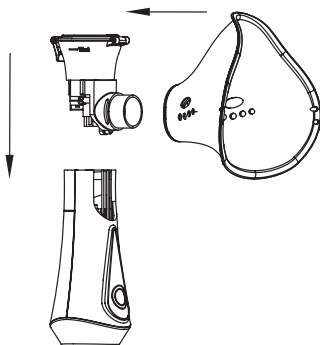

Using the Nebulizer

1. Press the power button to start the nebulizer; the indicator light will turn blue.
2. The nebulizer will begin operating at a high spray speed. Pressing the button again will switch to a lower speed. A third press of the button will stop the device.
3. Place the mask over your nose and mouth as shown in the figure below. Begin inhalation.
4. The nebulizer will turn off automatically when the medicine runs out. If you want to stop inhalation, press the button twice to turn off the device.
5. When the medication level is low, the indicator will flash blue for 2-3 seconds. After this time, the nebulizer will turn off.

Note: If no mist appears when the nebulizer is turned on, check if there is medication in the cup. If the medication cup is full, gently shake the device.

After Inhalation

1. After inhalation, pour out any remaining medication from the cup to avoid bacterial growth, which can negatively affect the device's performance. Unremoved medication residues can clog the mist spray hole.
2. Disconnect the mask/mouthpiece.
3. After each use, clean the medication cup, mouthpiece, and mask thoroughly with warm water. Leave to dry. More information on cleaning the nebulizer components can be found in the chapter: Cleaning and Storage.
4. Store in a dry and clean place.

Self-Cleaning Mode

The nebulizer is equipped with a self-cleaning mode. To activate it, pour a small amount of water into the medication cup, then hold the power button for 5 seconds. The blue indicator light will start flashing, and the device will emit mist in pulses for 10-15 seconds, then turn off.

Cleaning and Storage

It is very important to regularly clean and disinfect the nebulizer to maintain its performance and proper operation.


Cleaning outside of the Nebulizer:

Clean outside of nebulizer with dry wipes.

Cleaning the Medication Cup:

- Do not place the medication cup in the dishwasher.
- To clean the medication cup, press the button to detach it (see figure below). Submerge it in water or a solution of water with regular vinegar for 15 minutes, then thoroughly rinse the medication cup and leave it to dry.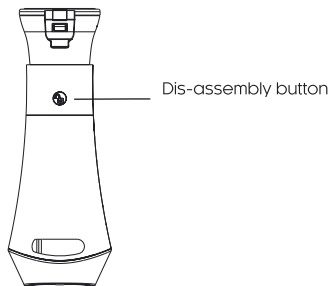

Cleaning the Silicone Mask/Mouthpiece:

- In the dishwasher: Place the mask/mouthpiece on the top rack of the dishwasher, using standard settings. Mineral deposits may appear after washing.
- By hand: Detach the mask/mouthpiece from the nebulizer, wash with warm water.
- Use a soft cloth or brush to remove any residue. Leave the mask/mouthpiece to dry.

Storage

- Turn off the nebulizer, disconnect the power cord, and empty the medication cup. Clean the nebulizer components according to the instructions.
- Ensure each part is completely dry.
- Place the nebulizer in a dry and cool place, such as a bag.

Troubleshooting

Problem	Solution
Indicator flashes yellow	• Charge the nebulizer. Connect the power cord and charge the device for 3-4 hours. • Make sure to use a working power plug.
No mist	• Wait 5 to 10 seconds after turning on the nebulizer. • If there is still no mist after 10 seconds, gently shake the device. • Check if there is liquid in the medication cup and remove any excess water from the mouthpiece/mask of the nebulizer.
Bad smell	• Clean the medication cup with a solution of water and white vinegar, then rinse thoroughly with water. Leave the empty medication cup in a cool place for 12 hours without connecting it to the nebulizer.
Nebulizer does not work	• Ensure the nebulizer is fully charged. If not, charge the device for at least 3-4 hours. • Contact Feelo service.

Specification

Product Name	Feelo Family Ultrasound Nebulizer
Class and type of device	The nebulizer complies with Directive 93/42/EEG, Annex V. Class II device, type BF application part.
Nebulization rate	~0.25 ml/min
Particle size	2-5 µm
Medication cup capacity	15 ml
Residual medicine volume	≤0,1 ml
Low medication level indicator	Indicator flashes blue
Frequency	110 kHz
Power supply	USB-C cable - Adapter (plug) 5V - 1A
Input rated voltage	DC 5V
Input rated power	1.5V
Rated working voltage	3.7V DC
Rated working power	1.0 W

Battery	Li-ion battery
Working time	Up to 120 minutes on a single charge
Charging time	3-4h
Battery-low alert	Indicator flashes yellow
Noise level	≤30 dB
Operating modes	Two spray speeds: low and high
Operating conditions	Ambient temperature: 5 ~40 Relative humidity: ≤90% Atmospheric pressure: 70 kpa~106 kpa
Storage/Transportation conditions	Ambient temperature: -40 ~55 Relative humidity: ≤90% Atmospheric pressure: 70 kpa~106 kpa
Dimensions	(L)58 mm x (W)51 mm x (H)137 mm
Weight	140 g
Water resistance class	IP22

Contraindications

Using an ultrasound nebulizer, although often effective and convenient, may not be suitable for everyone. Contraindications for the use of this device may include:

Health conditions

- Individuals with heart disease: Ultrasound waves can affect heart function, so those with serious cardiac issues should consult a doctor before use.
- Individuals with respiratory failure: In some cases, another type of nebulizer may be preferred.
- Individuals with epilepsy: The vibrations and sounds generated by the nebulizer can potentially trigger a seizure.

Allergies

- Some individuals may be allergic to the materials from which the nebulizer is made, which can cause allergic reactions.

Contamination and Hygiene

- Nebulizers, if not properly cleaned, can become a source of infection. Individuals with weakened immune systems should take special care to maintain the hygiene of the device.

Technical Condition of the Device

- Using a damaged nebulizer can be dangerous and ineffective.

Electromagnetic Compatibility

The Feelo Family nebulizer has been designed in accordance with current electromagnetic compatibility standards to ensure safe and effective use without mutual interference with other electronic devices. It is recommended to maintain an appropriate distance from devices emitting radio and microwave waves, such as Wi-Fi routers, mobile phones, and microwaves, to avoid possible interference. To minimize the risk of interference, do not use the nebulizer in close proximity to strong sources of electromagnetic radiation.

Warnings! Read before use

- For home use only.
- Consult a doctor before use if you are pregnant, breastfeeding, or have asthma.
- Individuals suffering from excessive sweating and pulmonary cough should consult a doctor before using this product.
- In case of skin irritation or eye contact with the medication, wash the affected area with water and consult a doctor. Stop using the product if you feel any discomfort.
- The product is not intended for self-repair.
- Store the charging cable in a dry and safe place.
- Straighten the cable before use.
- In case of malfunction, do not disassemble the device or replace any compo-

nents, as there is a risk of electric shock. Contact the manufacturer.

- A damaged power cord must be replaced with a new one. Contact the manufacturer.
- Do not expose the product to prolonged temperatures or sunlight. Do not place the nebulizer near a heat source.
- Do not place the nebulizer or its components in a microwave oven.
- Do not add any other medications to the nebulizer except those intended for nebulization. Do not use distilled water, essential oils, or sticky liquids in the nebulizer.
- After each use, pour out any remaining medication from the cup. Always use a fresh dose of medication.
- Always use a clean silicone mask/mouthpiece for inhalation.
- Do not use hot water to fill the medication cup.
- Ensure that the medication cup is properly mounted on the nebulizer.
- Make sure the medication cup is not empty while using the nebulizer.
- Store the nebulizer in a cool place.
- Do not cover any outlet of the device or insert objects into any opening.
- The nebulizer requires regular cleaning (see: Cleaning and Storage). Do not immerse the entire device in water while cleaning. Clean it with a dry cloth.
- Do not attempt to empty or fill the medication cup while the nebulizer is running.
- Do not use detergent to wash any part filled with water, as it may interfere with the nebulizer's performance.
- Disconnect the nebulizer from the power source before filling the medication cup.

Danger warning

To reduce the risk of shock:

- Always disconnect the device from the power source after use.
- Do not use during bathing.
- For more precise instructions on storing and using the device and included accessories, refer to the manufacturer.

To reduce the risk of fire, short circuit, or injury:

- Disconnect the charger from the power source after charging is complete.
- The product is not a toy. Exercise particular caution if the nebulizer is used by or near a child.
- Use the device only as intended. We recommend using accessories mainly recommended by the manufacturer.
- Do not use the charger if the power cord or plug is damaged. If the nebulizer does not function properly after being dropped or left in the open air, return it to the place of purchase.
- Do not pull, cut, or wrap the power cord around sharp edges. Do not operate the nebulizer while charging. Keep the device components away from heated surfaces.
- Do not disconnect the charger by pulling the cable. Use the plug.
- Do not handle the plug with wet hands.
- Never use the product if you have concentration issues or feel drowsy.

- Do not use the device outdoors or in areas where it may be exposed to aerosols or pure oxygen.
- Plug the device into a power outlet. Check its specifications.

Use of the device is not recommended for children under 3 years old without consulting a doctor. Children and individuals with limited sensory, motor, or mental abilities should use the device under the supervision of a caregiver or be properly trained on the correct and safe use of the device. Children should not play with the device. Children should not perform cleaning and maintenance of the device without supervision.

TWO-YEAR WARRANTY

The manufacturer provides a warranty for a period of two years from the date of purchase. The warranty is valid in Poland and within the territories of official distributors. The warranty covers damages related to manufacturing defects or improper workmanship of the product. Keep the proof of purchase for the duration of the warranty period. During the warranty period, you can file a claim if the product was used correctly and according to its intended purpose.

THE WARRANTY DOES NOT COVER:

Surface damages resulting from normal use.

Damages resulting from:

- a. Accidental drops or impacts, improper use, abuse or commercial use, neglect, modification of the device,
 - b. Improper maintenance, lack of care for the product,
 - c. Connection to an incorrect power source,
 - d. Repairs or attempts to repair by a party not authorized by the manufacturer.
- Any attempts to open or disassemble the device, its parts, or accessories will void the warranty.

Warranty claims must be supported by evidence. The claim must be made within the warranty period. The material covering the handle is delicate, so care should be taken during use. Scratches or abrasions on the surface of the handle will be treated as surface damages resulting from normal use. Therefore, they will not be covered by the warranty. The warranty applies to components that affect the functioning of the nebulizer. Proper care of the nebulizer significantly increases its lifespan. Failure to follow the recommendations may result in the loss of the warranty.

Nebulizer – Suitable Types of Medication

1. Suitable Medications and Their Usage Principles

A nebulizer atomizes liquid medication into fine particles (usually 1-5 μ m) through a high-frequency vibrating microporous network. Therefore, the physical properties of the medication (viscosity, solubility, particle size) must meet specific requirements.

A. Bronchodilators

- Examples of medications: salbutamol, ipratropium bromide, levosalbutamol.
- Usage principles:
 - ▶ Water-soluble solutions with low viscosity (similar to normal saline).
 - ▶ Do not leave residue and can pass through the microporous membrane smoothly.
- Concentration requirements:
 - o Salbutamol: The standard concentration is 0.5% (5 mg/ml); it should be diluted to 2.5 mg/2.5 ml (normal saline).
 - ▶ Ipratropium bromide: Use in the special atomized form (0.25 mg/ml).

B. Inhaled Glucocorticoids

- Example medication: budesonide suspension (Pulmicresol).
- Usage principles:
 - ▶ The suspension must be specially processed into ultrafine particles (<2 μ m) and requires dilution before use.
- Concentration requirements:

- ▶ Budesonide: Dilute to 0.25-1 mg/ml (e.g., 1 mg budesonide + 2 ml normal saline).
 - C. Normal Saline/Sterile Water for Injection**
 - Usage principles:
 - ▶ Isotonic, non-viscous, used only for drug dilution or airway humidification.
 - Concentration requirements:
 - ▶ Normal saline 0.9% (for direct use).
 - D. Mucolytics**
 - Example medication: acetylcysteine.
 - Usage principles:
 - ▶ Low concentration is required (high concentration has too high viscosity).
 - Concentration requirements:
 - ▶ 10% acetylcysteine solution should be diluted to 3-5% (e.g., 3 ml drug + 3 ml normal saline).
- 2. Medications That Are Not Suitable for a Mesh Nebulizer**
- A. Oily Medications (e.g., fat-soluble vitamins, essential oils)**
 - Reasons for exclusion:
 - ▶ Oily liquids do not form aerosols through the microporous membrane and cause clogging.
 - ▶ Inhalation may lead to lipid pneumonia (lungs cannot metabolize oils).
 - B. Non-specialized Suspensions (e.g., regular dexamethasone solution)**
 - Reasons for exclusion:

- ▶ Large particles (>5µm) cannot pass through the membrane and deposit, blocking the mesh.
- ▶ Excipients such as phenol may damage nebulizer components.
- C. Hypertonic Solutions** (e.g., 10% NaCl, 50% glucose)
 - Reasons for exclusion:
 - ▶ High osmotic pressure increases viscosity and reduces nebulization efficiency.
 - ▶ Risk of crystallization (e.g., high concentrations of sodium chloride).
- D. Chinese Herbal Injections** (e.g., Houttuynia, Shuanghuanglian)
 - Reasons for exclusion:
 - ▶ Contain plant proteins, polysaccharides, and other macromolecular substances that can clog the membrane.
 - ▶ May trigger allergic reactions.
- E. Medications Containing Preservatives/Thickeners** (e.g., benzalkonium chloride, propylene glycol)
 - Reasons for exclusion:
 - ▶ Preservatives deposit on the membrane surface, reducing nebulization efficiency over time.
 - ▶ Propylene glycol increases the viscosity of the liquid and affects nebulization performance.

3. Medications That Easily Clog the Membrane and Their Causes

Medication Type	Cause of Clogging
Undiluted Budesonide	Suspension concentration is too high (1 mg/ml undiluted), high viscosity, and particles easily settle.
Acetylcysteine Solution	High viscosity of 10% solution, sulfur compounds easily oxidize and form crystals.
Cough Syrup	Contains sucrose or honey; sugar crystals stick to the mesh; thickeners like xanthan gum increase viscosity.
Proteins (Insulin)	Protein denatures and coagulates during vibration, forming a gel that blocks the micropores.
Unshaken Suspension	Drug particles settle, local concentration becomes too high, and large particles directly impact the membrane during nebulization.

4. Compatibility of Drug Concentration and Nebulizer

- Budesonide: Must be diluted to 0.25-1 mg/ml (direct use of the original solution can easily clog membrane holes).

- Acetylcysteine: Recommended concentration is 3-10%; some models require dilution to less than 5%.
- Tobramycin: Only available in a special atomized dosage form (60 mg/ml, no dilution required).
- Salbutamol: The standard concentration of the nebulized solution is 0.5%, requiring a 1:1 dilution (to reduce irritation).

5. Principles of Scientific Selection

- Viscosity Requirements: The viscosity of the liquid medication should be close to normal saline (1-3 mPa·s), and fluidity can be tested using a syringe.
- pH Range: The pH of the liquid medication should be 4.5-7.5; excessive acidity or alkalinity may damage the membrane.
- Particle Size: The suspension should be clearly labeled as „atomizing special,” and the median particle diameter (MMAD) should be $<3\mu\text{m}$.

6. Maintenance Recommendations

A. Preparation Before Cleaning

- Tools/Materials:
 - ▶ Warm water (around 50°C, no more than 60°C to avoid damage).
 - ▶ Soft brush for cleaning the nebulizer.
 - ▶ Neutral detergent (e.g., baby bottle cleaner).
 - ▶ Sterilized water for rinsing.
 - ▶ Cotton swabs or a soft microfiber cloth.

- ▶ 75% medical alcohol (only if allowed by the manual).
- Device Shutdown:
 - ▶ Turn off the nebulizer and remove the battery (if applicable).
 - ▶ Disassemble the atomizing cup and mesh according to the instruction manual.

B. Step-by-Step Cleaning Method

- 1. Initial Rinse** – Rinse the membrane with warm water to remove visible drug residues.
- 2. Soaking** – Soak the membrane in warm water with a few drops of detergent for 10-15 minutes (or 30-60 minutes for oily residues).
- 3. Gentle Brushing** – Use a soft brush in circular motions.
- 4. Thorough Rinse** – Rinse the membrane with sterilized water to remove detergent residues.
- 5. Disinfection and Drying** – If necessary, wipe the back of the membrane with an alcohol-soaked cotton swab or use steam.

C. Special Methods for Stubborn Contamination

- **Enzyme Detergent Soak** – Use a cleaning solution containing protease (e.g., contact lens cleaning solution), dilute at 1:10, and soak for 1 hour.
- **Ultrasonic Cleaning Machine** – Place the nebulizer in warm water and run the low-frequency mode for 3-5 minutes (ensure the device is water-resistant).
- **Weak Acidic Solution** – Mix white vinegar with water at a 1:3 ratio, soak for 5 minutes, then rinse immediately (only for stainless steel mesh).

D. Functional Test After Cleaning

- No-Load Test:
 - ▶ Assemble the nebulizer, add 2-3 ml of normal saline, turn it on, and check if the mist is uniform.
 - ▶ Normal condition: continuous mist, no intermittent splashing sounds.
- Clogging Assessment:
 - ▶ If mist output is significantly reduced or delayed, repeat cleaning or replace the mesh.

E. Regular Maintenance Recommendations

- Preventive Cleaning – Rinse the mesh with warm water immediately after each use.
- Deep Cleaning – Soak once a week with a neutral detergent.

F. Risk Warnings

- Prohibited Cleaning Agents:
 - ▶ Strong acids (e.g., toilet cleaners), strong alkalis (e.g., bleach), hydrogen peroxide (damages metal mesh).
- Avoid High Temperatures:
 - ▶ Most meshes can withstand up to 80°C; boiling water immersion or high-temperature steam can cause permanent damage.
- Replacement Instead of Repair:
 - ▶ If the mesh is visibly damaged or the coating peels off, replace it immediately.

By following these steps, most issues caused by improper drug use can be effectively prevented. If the nebulizer still does not function correctly after cleaning, contact the manufacturer to purchase an original mesh, as non-original parts may affect ne-

bulization efficiency.

Tip: If contraindicated medications (e.g., antibiotics, hypertonic solutions) need to be used, it is recommended to use a compression nebulizer (jet flow), which has broader compatibility with medications.

The product meets the requirements of the so-called „New Approach” Directives of the European Union (EU).

RoHS

RoHS stands for Restriction of Hazardous Substances Directive. It means that the product does not contain hazardous substances such as lead, mercury, cadmium, hexavalent chromium, PBB, and PBDE in amounts exceeding permissible limits.

This symbol indicates that the product is patented and copying it is prohibited. It protects the intellectual property rights of the manufacturer.

Exercise caution when using the product.

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This device is marked in accordance with the Polish Act on Waste Electrical and Electronic Equipment and the WEEE Directive 2012/19/EU with the symbol of a crossed-out waste bin. This marking indicates that this equipment, after its service life, cannot be disposed of with other household waste. The user is required to hand it over to those conducting the collection of waste electrical and electronic equipment. Those conducting the collection, including local collection points, stores, and municipal units, create an appropriate system that enables the disposal of this equipment. Proper handling of waste electrical and electronic equipment helps to avoid harmful health and environmental consequences arising from the presence of hazardous components and improper storage and processing of such equipment.

The product is recyclable and should be sorted and processed in accordance with local recycling regulations.
Product made in China.

For domestic use only.

Manufacturer: Shenzhen Jermei Medical Device Technology Co., Ltd. Room 201, 202, 4th Bldg XianYuXing Company, 4th Bldg XianYuXing industrial Zone, Yuhe Road, Gonghe Community, Shajing Street, Baoan District, Shenzhen, P.R. China

Importer: SHOP DENT Sp. z o.o.
ul. Dziupli 12, 02-454 Warsaw, Poland
complaint service: reklamacje@shop-dent.pl
customer service: kontakt@feelo.pl
product safety: kontakt@feelo.pl

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Luxus Lebenswelt GmbH